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## Circulaire 7584

du 15/05/2020

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## INSTRUCTIONS

Description: community masks, pleated, non-sterile and meeting the requirements of standard NBN / DTD S 65-001: 2020 "Community masks and handmade masks - Guide to minimum requirements, making, maintenance and use"

Duration of use and precautions for use: single use for 4 hours maximum and throw away after use. If a mask is damaged or at the end of its life, it can, after having been washed, be thrown away with the usual waste. Unwashed masks should be wrapped in a closed plastic bag and then discarded in a closed bin. If a bin for medical waste is available, an unwashed used mask can be discarded.

Warnings: This product is not a medical device within the meaning of Regulation (EU) 2017/745 & Directive 93/42 / EEC or personal protective equipment within the meaning of Regulation (EU) 2016/425.

This product can only be used by children over 3 years of age due to the risk of suffocation and children should be supervised when wearing this product. Wearing a mask requires adaptation, namely the feeling of having less air, this is normal, the mask works. Please note: a mouth mask certainly does not allow intense effort, like intensive sport. This product is not designed to protect the lungs from hazards in the air. It is designed to avoid touching your mouth and nose and to minimize saliva splashes when talking, coughing or sneezing. The mask absolutely does not exempt the wearer from the application of barrier gestures supplemented by the measure of social distancing which are essential.

  
**MASKS FOR BELGIUM**  
[www.masksforbelgium.com](http://www.masksforbelgium.com)

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## EN: Instruction manual for correct use of your mouth-nose-mask for everyday use

 Please read carefully before first use!

1. Wash your hands with soap BEFORE removing the mask(s) from the packaging
2. **IMPORTANT:** Wash the mask(s) at 60C° for 30 minutes BEFORE the first use.
3. Wash your hands AFTER putting the mask(s) in the washing machine for washing. After washing, air dry the mask(s) until they are completely dry. Before using the mask(s), wash your hands with soap.
4. Place a paper filter in the pocket between the 2 layers of fabric. You can use a piece of kitchen paper, toilet paper or a single layer of coffee filter. Sanitary towels are not recommended because of the chemicals they contain.
5. Place the mask in front of the face so that the nose, mouth and chin are covered. The thinner fabric (fleece) is used as the inner side. Tie the two upper fabric straps at the upper back of the head with a tight knot. Then knot the two lower fabric bands together at the back of the neck.
6. Make sure that the mask lies directly on the naked skin and that no hairs are trapped underneath. Make sure that the mask **ALWAYS** covers the nose, mouth and chin. Do **NOT** touch the mask while wearing it. Do **NOT** wear the mask under the chin at the neck whilst not using.
7. Wash your hands with soap every time you touch your mask.  
TO REMOVE THE MASK: Unknot the bow at the neck. Do not unknot the tight knot behind the head. Take off the mask with both hands by pulling the mask far away from the face over the head forward. Make sure that the mask does not touch your face.
8. Remove the paper filter and dispose of it in a trash can with a lid. Place the mask(s) with the tied upper straps in a plastic bag for storage until the next wash. The solid upper head knot can remain knotted even during washing, making it easier to adjust to your head in the future. Wash your hands with soap.
9. Wash the mask(s) without the plastic bag:
  - a. either in a sufficiently large container at 70-100C° with some liquid detergent, stirring it with a wooden spoon. Leave the mask(s) in hot water for 30 minutes, add new hot water if necessary. Rinse with hot clear water and hang the mask(s) to dry until completely dry.
  - b. Or wash the mask(s) in the washing machine at 60C° for 30 minutes.
 Always wash your mask(s) the evening of the day you wore them. You can wash several masks at the same time as long as these are masks of people living in the same household. **DON'T** iron the mask(s) as they are made of synthetic fabric. Always wear your own mask(s).
10. Wash your hands with soap.
11. Always wear your own mask(s) and don't wear the same mask for more than 4 hours.
12. Keep a distance of at least 2 metres to people who do not live with you in your household. If you wear glasses, make sure that the lower edge of your glasses is on the upper edge of the mask and not pinched behind them. This will prevent the glasses from fogging up.